wellbee 
by ZiboCare

JUNIOR

WEIGHTED DUVET

- EN** Instructions for use - 2
 - LV** Lietošanas instrukcija - 11
 - LT** Naudojimo instrukcijos - 20
 - ET** Kasutusjuhend - 29
-

Wellbee Junior weighted duvet

Congratulations on your Wellbee Junior weighted duvet by ZiboCare®. You have chosen a quality product by ZiboCare®, specially developed for JYSK based on our many years of experience in the field and in collaboration with both users and healthcare professionals.

We hope that you will have better sleep and well-being with your Wellbee Junior weighted duvet.

Sensory stimulating

Wellbee Junior is a sensory stimulating weighted duvet providing deep pressure touch. The pressure and weight of the duvet have a calming effect which helps with falling asleep faster, fewer nighttime awakenings, and deeper, more undisturbed sleep as well as relaxation during the day.

The weight of the duvet is created by evenly distributed glass beads sewn into quilted squares, which provide an even and enveloping pressure. The size and smooth surface of the glass beads make it a silent filling that easily adapts to your movements and allows the duvet to fit close to your body.

Ventilating and temperature-regulating

The bolster of the weighted duvet is made of 100% viscose, made from bamboo, a soft and breathable material known for its temperature-regulating and moisture-wicking properties ensuring an optimal sleep environment.

Washable

The weighted duvet is designed for easy care, as it can be machine-washed at 60°C, making it a practical and convenient choice for everyday use.

IMPORTANT KEEP FOR FUTURE REFERENCE

To fully benefit from this product and ensure safe usage please read this User Manual before use and save it for future reference.

Scope of use

Intended purpose

Wellbee Junior is a weighted duvet designed to provide deep pressure touch.

Clinical benefit

The deep pressure touch helps to regulate the nervous system and promote a sense of calm and comfort.

The weighted duvet can be used to promote relaxation and help alleviate the users' difficulties with falling asleep, interrupted sleep, or poor sleep quality.

Intended user

The intended user of the Wellbee Junior weighted duvet is children over 3 years.

The user must be psychically and mentally capable of removing the weighted duvet if used without assistance.

The user can be assisted by a lay person or healthcare professional as it does not require special training or occupational skills to handle the weighted duvet.

The users could be, but are not limited to, children with sleep problems. The weighted duvet can be used by anyone (over 3 years old) who thinks they benefit from the weighted duvet. Users must be aware of limitations and contra-indications.

Intended use environment

The weighted duvet is for indoor use only.

Safety information

Limitations

Junior duvets are recommended for cots from 600-1200 mm.

The weighted duvet can be tucked in but must not be rolled around the body.

The weighted duvet must not be used to fixate the user.

Contra-indications

If the user has intellectual disabilities or cannot communicate their experience with the weighted duvet, it must be used under supervision.

In case of circulatory, respiratory or musculoskeletal conditions or if the user has implanted device, a healthcare professional must be consulted before use.

Precautions

Ensure there is enough space around the user to move the weighted duvet without difficulty.

Consider the thermal resistance of the weighted duvet, the room temperature and the child's sleepwear.

Please be aware that the weighted duvet fits close to the body, and this can retain the heat from the body, which is inappropriate when having a fever.

Avoid sharp and pointed objects when using the weighted duvet as they can damage the product.

Check that the weighted duvet is intact before use and wash, and that there is no visible damage or tears in the fabric or seams.

Warnings

-  Must only be used for the intended use as described in the User Manual.
-  Not for children under 3 years.
-  Ensure the child's face is uncovered.
-  Contains small parts (glass beads) which could present a health risk if removed from their intended placing inside the duvet.
-  Overheating can endanger the life of your child!
-  Do not use the weighted duvet folded.
-  Do not roll the weighted duvet around the body.
-  Do not use in combination with sleeping bag, blanket or other weight products.
-  Keep away from fire. Avoid using the weighted duvet near fire sources, such as fire, hot surfaces, burning cigarettes or hot lamps, as the product is not resistant to ignition.
-  Stop using and washing the weighted duvet at the first sign of damage.
-  Observe the surface temperature on the product, before placing it on a user. This goes especially for users with insensitive skin as they cannot feel the heat. If the surface temperature is above 41°C, then allow the product to cool before use.
-  The weighted duvet can pose a trip hazard if lying on the floor.
-  The bag handles can pose an entrapment hazard if used as a toy.

How to use the Wellbee Junior

The weighted duvet can be used in a lying or sitting position instead of a regular duvet.

To protect the weighted duvet during use, it is recommended to place it in a regular duvet cover.

Storage and transportation

To protect the weighted duvet, store and transport it in the supplied bag. Fold the duvet in half twice in each direction to make it fit easily into the bag.

Ensure the duvet is kept dry and out of direct sunlight during storage and transportation.

Care instructions

Cleaning

It is recommended to shake and air the weighted duvet regularly. The weighted duvet cannot withstand being beaten or vacuumed.

Washing

The need and frequency of washing depends on use and application.

Check the capacity of the washing machine before washing.

See the explanation of care symbols in the symbol explanation table.

Maintenance

The weighted duvet does not require any maintenance other than as stated for cleaning and washing.

Lifetime

The expected lifetime of the weighted duvet is 5 years, assuming proper use, handling, washing, and storage as outlined in the instructions. Use of the product beyond its expected lifetime is the user's responsibility and at their own risk.

Recycling and disposal

Waste is sorted and disposed of according to local rules.

Warranty (Returns and complaints)

In the event of a return or complaint, please contact the point of purchase or the location where the product was obtained. They will assist with the further handling of the case in accordance with the applicable terms and conditions of sale.

The warranty covers manufacturing defects. The warranty does not cover normal wear and tear, deterioration of the product or damage resulting from careless handling or care of the product.

Serious incidents

According to the authorities' regulations regarding medical devices, it is the duty of the manufacturer and healthcare personnel to report accidents and incidents involving their products.

In case of any accidents or incidents relating to the product please report it to ZiboCare immediately.

CE Declaration of Conformity

This product conforms to the requirements of the Medical Device Regulation 2017/745.

Technical specifications

Thermal resistance: 3.0 togs.

Materials:

Bolster:

100% viscose, made from bamboo, FSC certified OekoTex Standard 100 Class I.

Filling: 90% glass beads, 7.5% polyester fibers, 2.5% polypropylene fibers.

Filling weight (glass beads): 2394 g

Variants

Model	REF	Size (mm)	Weight (kg)
Wellbee Junior by Zibocare®	4063339JB	1000x1400	3

Labelling

The weighted duvet is labelled with important product specific information such as production date, batch code and catalogue number that should be kept for future reference.

Symbol explanation

	Warning , see Important safety instructions
	Consult instructions for use (User Manual). The latest version of the Instruction for use can be downloaded as a “pdf-file” from the www.jysk.com web page. The manuals can be read with a “Read Out Loud” functionality in e.g. Adobe Acrobat and /or printed in a larger size to accommodate persons with visual impairments. The user information is not available in Braille.
	Batch code
	Catalogue number
	Date of manufacture
	Manufacturer
	Distributor
	Medical device
	Unique device identifier
	CE mark
	Keep plastic covering away from children to avoid suffocation
	Keep away from rain
	Keep away from sunlight

	Mild washing, maximum 60°C
	No bleaching
	Flat drying
	No tumble drying
	No ironing
	No dry cleaning
	For indoor use only
	Intended purpose. Sensory stimulant

Pre-sale information

Information	User Manual section
Where to find information readable for users with visual, reading and cognitive impairments	Symbol explanation
Intended use	Scope of use
Intended user	Intended user
Intended environment	Intended use environment
Maintenance instruction	Care instructions
Cleaning instruction	Care instructions
The overall dimensions and mass	Variants
Dangerous combinations	Warnings
Foldable / dismantlable	Storage and transportation
Transport instructions (e.g. in a car or aeroplane)	<i>No restriction on means of transport</i>
Warnings	Warnings
Guidance (thermal resistance, room temperature and the child's sleep-wear)	Precautions
Cot dimensions	Limitations

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wellbee

by ZiboCare

The Wellbee weighted duvet is also available for adults.

For more information see www.byzibocare.com.

A black kettlebell with a silver handle is positioned on the left side of the frame, resting on a stack of white towels. The kettlebell's handle is curved and metallic, showing some wear. The main body of the kettlebell is black and has the letters "KG" printed in white. The stack of towels is neatly folded, showing a ribbed texture. The background is a dark, solid color, possibly a dark grey or black, which makes the white towels and the black kettlebell stand out.

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ZiboCare A/S